

Dementia C.A.R.E. Alarm

(Ceiling and remote equipment)



Locating the CARE Alarm:

- Fit the CARE alarm using either the wall bracket or belt clip as required (i.e. wall, door, chair bracket).
- Make sure that the unit has good visibility of the Infra Red Ceiling Unit. If unsure then activate the unit using the appropriate method before installing.

Activating the CARE Alarm with a Pressure Mat:

- Insert a Jack to Jack lead with one end into the socket on the side of the unit, and the other into a pressure mat.



- Press the Blue button on the side of the unit to set the activation delay period as required. – This is the delay before the CARE unit goes into alarm once it has been activated.



The unit will beep to notify you of the delay programmed as follows:

<u>Press</u>	<u>Beeps</u>	<u>Delay</u> (approx)
1	1 beep	1 second
2	2 beeps	3 seconds
3	3 beeps	5 seconds (RECOMMENDED FOR MATS)
4	4 beeps	8 seconds (RECOMMENDED FOR DOORS)
5	5 beeps	16 seconds (RECOMMENDED FOR DOORS)

(after 5 presses it goes back to the first press)


18 The Arkwright Suite, Coppull Mill, Mill Lane, COPPULL PR7 5BW

Tel: 01257 794333 Fax: 01257 794777 Email: info@bleeperltd.co.uk

- Position the resident onto the chair or bed mat and press the yellow reset button.
- The CARE Alarm is self-arming. This takes approximately 15 seconds and occurs only when a resident is placed on a pressure mat, or when the door has been closed. The CARE alarm emits a series of 6 rapid beeps to indicate it has been armed.
- The CARE alarm is now armed. If the user gets off the pressure mat, the alarm will activate the alarm system. Whilst in alarm the CARE alarm will emit a beep and flash the red led once every 3 seconds.

Resetting the CARE Alarm:


- To reset the CARE alarm simply press the yellow reset button. The unit will emit 3 rapid beeps to acknowledge that the reset button has been pressed.

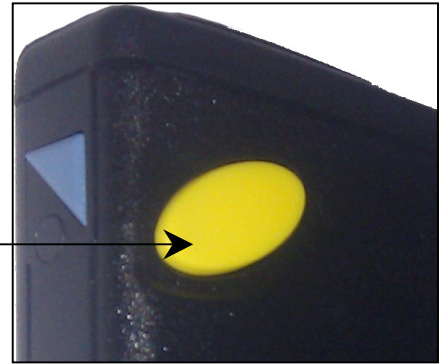

Activating the CARE Alarm with a Door Magnet:

- The CARE alarm is operated in the same way as described above with a jack to jack lead however the alarm must be positioned on the bracket **below** the door magnet.
- Press the Blue button to set the activation delay period as normal.
- Close the door and press the yellow reset button, and as before allow approximately 15 seconds for the unit to arm. This is indicated with a series of 6 rapid beeps.

Door Magnet

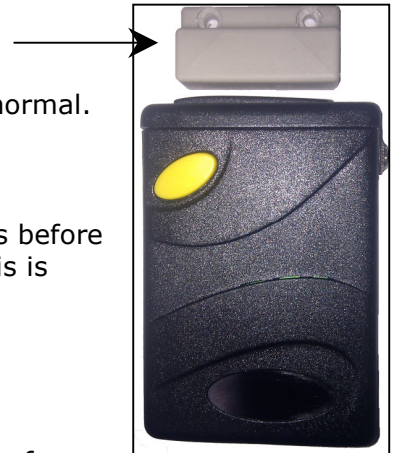

A high delay period is advised (4 or 5 beeps) as this allows time for the member of staff to enter the room, close the door behind themselves and press the yellow reset button before alarm activation occurs.

Changing the Battery:

The CARE alarm takes a 3.6V AA lithium battery. The battery cover can only be released by pressing the nib of a pen into the hole by the cover and sliding it off at the same time.

