




# LIVE-LINK MOVEMENT SENSOR



the smart choice

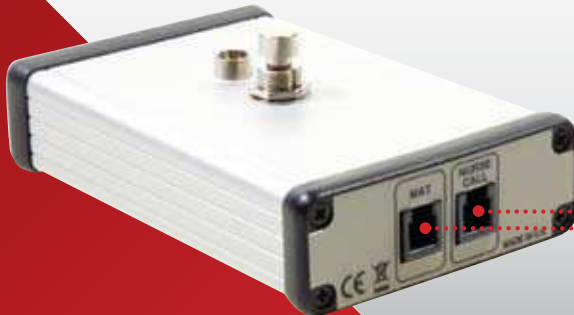

- PLUG IN YOUR NURSE CALL ALARM HERE
- PLUG IN A CHAIR PAD HERE



## PLUG AND PLAY

Will work with your Nurse Call system or your money back

## UNIQUE, FLEXIBLE, PORTABLE ONE OF A KIND

Dementia, frailty and confusion are amongst the most common reasons for resident injury, to name but a few. Different residents have specific needs which currently necessitates the purchase of a wide-range of peripherals.

Live-Link's Movement Sensor makes carers lives easier by allowing you to use one device for a multitude of purposes in one, robust, easy to use, reliable unit. It's one of a kind. The Live-Link Movement Sensor can be set to register a call system alert under a range of different needs.

Over the years we have listened to feedback from customers about how hard it seems to be to monitor residents at risk of falling. Unlike a pressure mat, our sensor can't be stepped over or avoided. We feel that the Live-Link Movement Sensor could put an end to these risks.

**BATTERY**

- LASTS UP TO 2 YEARS\* USER REPLACEABLE
- BATTERY-LOW ALERT

**DURABLE CONSTRUCTION**

- MADE OF HIGH-GRADE ALUMINIUM
- HEAVY DUTY FOOT SWITCH
- SPLASH PROOF

**DUAL PURPOSE**

- CAN BE USED WITH A CHAIRPAD DURING THE DAY
- ON BED TABLE AT NIGHT

**ADAPTABLE MOVEMENT SENSOR**  
TELLS YOU IMMEDIATELY

- WHEN A RESIDENT ATTEMPTS TO LEAVE THEIR BED
- WHEN A RESIDENT HAS LEFT THEIR BED
- WHEN A RESIDENT EXITS THEIR ROOM

\* based on 4 Duracell Industrial AA batteries and up to 10 activations per day  
\*\* minus postage costs. Based on return of goods in original packaging and condition. (Refund excludes P & P)  
\*\*\* Movement Sensor Only (does not cover faults arising out of neglect or careless handling)

**QUICK, EFFICIENT SOLUTIONS FOR  
CARER & PATIENT**

Configure it to alert you when a patient/resident:

- Sits up in their bed
- Leaves their bed
- Leaves their room
- Leaves their chair (chair sensor pad can be purchased separately)
- Set a time delay (2,5,10 or 15 minutes) for more independent residents that can perform certain tasks (like visiting their en-suite)

*A unique, easy to use unit that's reliable, portable and cost-effective.  
Reducing risk and maximising safety.*

# EFFORTLESS OPERATION

Live-Link's Movement Sensor is fully portable and can be moved very easily from one location to another and can be linked into the majority of nurse call systems. The unit comes pre-programmed with 4 x AA Batteries fitted and can be installed and operational within a few minutes.



Movement Sensor is powered by 4 x AA Batteries

**Battery Life:** 12 months to 3 years depending on usage and battery type

**Low-battery indicator:** The red LED on the Movement Sensor will be permanently lit and repeated calls are sent to the nurse call unit it is linked into to alert staff that new batteries are required. All four batteries need to be replaced at the same time.

Please note this piece of equipment is designed to compliment the care of the patient and not replace good nursing care.

Live-Link has a policy of continuous product development and reserves the right to change any prices and specifications given in this brochure.

## LIVE-LINK

YOUR COMPLETE NURSECALL ALARM SYSTEM



RECEIVER PANELS



TRANSMITTER UNITS



DECT PHONES



PERIPHERALS

FREEPHONE: **08080 687732**

E: [livelink@nursecallalarms.co.uk](mailto:livelink@nursecallalarms.co.uk)

[www.nursecallalarms.co.uk](http://www.nursecallalarms.co.uk)